

OPTIMIZED POINT-OF-CARE SOLUTION

MAKING TEST EASY

Getein1100

Immunofluorescence Quantitative Analyzer



Cardiac
Markers

Coagulation
Markers

Diabetes
Mellitus

Inflammation

Thyroid
Function

Metabolic
Marker

Renal
Function

Tumor
Markers

Reproduction
/Fertility

Infectious
Disease

Getein1100

Immunofluorescence Quantitative Analyzer



HIGHLY EFFICIENT & ACCURATE

Advanced fluorescence immunoassay

Multiple quality control



REAL-TIME AND RAPID TEST


One-step test

3-15 min/test

5 sec/test for multiple tests

OPERATION MODES

Inside Mode (single sample rapid test mode)

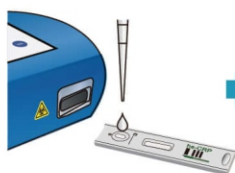

Sample Dispense



Test Card Insert



Click "Start" Icon



Result Show and Print

Quick mode (mass samples rapid test mode)



Sample Dispense



Timing the Reaction Manually



Click "Start" Icon



Result Show and Print



CONVENIENT OPERATION

RFID card calibration

Keyboard and mouse connectivity through USB port

Handwriting input available

Continuous test for 3 hours with lithium battery




USER-FRIENDLY INTERFACE

Android system

7 inch touch screen

FRONT VIEW

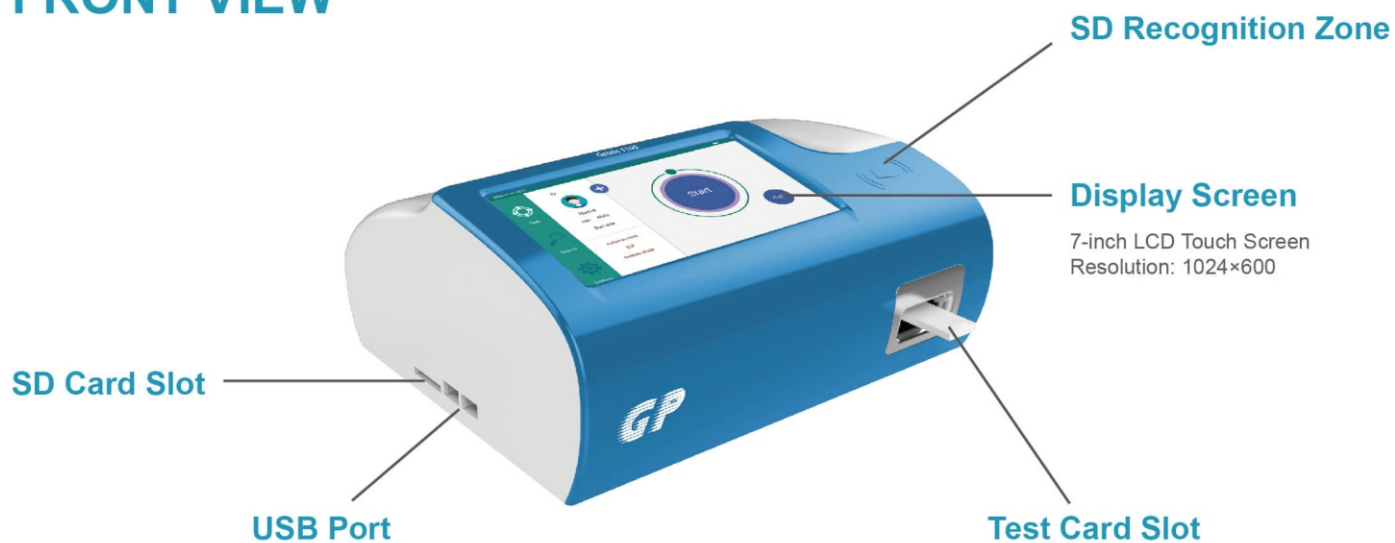



PORTABLE DESIGN

Small in size: 261 ×241 ×115 mm

Light in weight: 2.0 kg



LARGE MEMORY

Up to 10,000 results storage capacity

TECHNICAL PARAMETERS

Methodology

Immunofluorescence

Result

Quantitative

Sample Type

WB, plasma, serum, urine, Stool,
Nasal swab, Saliva, Capillary blood

Storage Capacity

10000 data

Language

English/Chinese/Spanish/Portuguese

Screen

7 inch touch screen

Power Supply

100-240 V~50 Hz/60 Hz, 60 VA

Working Environment

Temperature: 10-35℃
Relative humidity ≤70%
Air pressure 70.0~106.0 kpa

Dimension

261 mm×241 mm×115 mm (D×W×H)

Weight

2.0 kg

TEST ITEMS

Cat. #	TEST ITEMS	DISEASES	CUT-OFF VALUE	SAMPLE TYPES	MEASURING RANGE	SAMPLE VOLUME	REACTION TIME	QUALIFICATION
Cardiac Markers								
IF1001	cTnI	Myocardial infarction	0.10 ng/mL	S/P/WB	0.10-50.00 ng/mL	100 µL	10 min	NMPA CE
IF1098	TnT	Myocardial infarction	14.0 pg/mL	S/P/WB	10.0-10000.0 pg/mL	100 µL	15 min	NMPA CE
IF1089	BNP	Heart failure	100.0 pg/mL	P/WB	5.0-5000.0 pg/mL	100 µL	10 min	NMPA CE
IF1002	NT-proBNP	Heart failure	300 pg/mL	S/P/WB	100-35000 pg/mL	100 µL	10 min	NMPA CE
IF1005	CK-MB/cTnI/Myo	Myocardial damage /infarction	CK-MB: 5.00 ng/mL cTnI: 0.10 ng/mL Myo: 70.0 ng/mL	S/P/WB	2.50-80.00 ng/mL 0.10-50.00 ng/mL 30.0-600.0 ng/mL	100 µL	10 min	NMPA CE
IF1012	CK-MB/cTnI	Myocardial damage /infarction	CK-MB: 5.00 ng/mL cTnI: 0.10 ng/mL	S/P/WB	2.50-80.00 ng/mL 0.10-50.00 ng/mL	100 µL	10 min	CE
IF1014	H-FABP	Myocardial damage	6.36 ng/mL	S/P/WB	1.00-120.00 ng/mL	100 µL	3 min	NMPA CE
IF1016	CK-MB/cTnI/H-FABP	Myocardial damage /infarction	CK-MB: 5.00 ng/mL cTnI: 0.10 ng/mL H-FABP: 6.36 ng/mL	S/P/WB	2.50-80.00 ng/mL 0.10-50.00 ng/mL 2.00-100.00 ng/mL	100 µL	10 min	NMPA CE
IF1018	CK-MB	Myocardial injury	5.00 ng/mL	S/P/WB	2.50-80.00 ng/mL	100 µL	10 min	CE
IF1078	ST2	Heart failure	35.0 ng/mL	S/P/WB	3.1-200.0 ng/mL	100 µL	10 min	CE
Coagulation Markers								
IF1006	D-Dimer	Venous thromboembolism	0.50 mg/L	P/WB	0.10-10.00 mg/L	100 µL	10 min	NMPA CE
Inflammation								
IF1003	hs-CRP+CRP	Cardiovascular inflammation /normal inflammation	3.0 mg/L 10.0 mg/L	S/P/WB/ Fingertip blood	0.5-200.0 mg/L	10 µL	3 min	NMPA CE
IF1007	PCT	Sepsis, bacterial infection	0.10 ng/mL	S/P/WB	0.05-50.00 ng/mL	100 µL	15 min	NMPA CE
IF1015	PCT/CRP	Sepsis, bacterial infection	PCT: 0.10 ng/mL CRP: 3.0 mg/L	S/P/WB	0.10-50.00 ng/mL 0.5-200.0 mg/L	100 µL	15 min	NMPA CE
IF1044	SAA	Bacterial/Virus infection	10.0 mg/L	S/P/WB/ Fingertip blood	5.0-200.0 mg/L	10 µL	5 min	NMPA CE
IF1090	SAA/CRP	Neonatal sepsis, Bacterial/virus infection	SAA: 10.0 mg/L CRP: 10.0 mg/L	S/P/WB/ Peripheral blood	5.0-200.0 mg/L 0.5-200.0 mg/L	10 µL	5 min	NMPA CE
IF1088	IL-6	Acute inflammation	7.0 pg/mL	S/P/WB/ Peripheral blood	1.5-4000.0 pg/mL	100 µL	15 min	CE
Renal Function								
IF1008	CysC	Acute and chronic renal diseases	0.51-1.09 mg/L	S/P/WB	0.50-10.00 mg/L	10 µL	3 min	NMPA CE
IF1009	mAlb	Diabetic nephropathy, hypertensive nephropathy	20.0 mg/L	Urine	10.0-200.0 mg/L	100 µL	3 min	NMPA CE
IF1010	NGAL	Acute kidney injury	Serum: 200.0 ng/mL Urine: 100.0 ng/mL	S/Urine	50.0-5000.0 ng/mL	10 µL	10 min	NMPA CE
IF1011	β ₂ -MG	Acute and chronic kidney diseases/tumours	0.80-3.00 mg/L	S/P/WB	0.50-20.00 mg/L	10 µL	3 min	NMPA CE
Diabetes Mellitus								
IF1017	HbA1c	Diabetes mellitus	3.80%-5.80%	WB	2.00%-14.00%	10 µL	5 min	NGSP NMPA IFCC CE
Metabolic Marker								
IF1031	25-OH-VD	Osteomalacia, osteoporosis	30.00-50.00 ng/mL	S/P	8.00-70.00 ng/mL	200 µL	15 min	NMPA CE
Thyroid Function								
IF1024	TSH	Thyroid malfunction	0.27-4.20 µIU/mL	S/P	0.10-50.00 µIU/mL	100 µL	15 min	NMPA CE
IF1022	T3	Hyperthyroidism, hypothyroidism	1.30-3.10 nmol/L	S/P	0.30-10.00 nmol/L	40 µL	15 min	CE
IF1023	T4	Hyperthyroidism, hypothyroidism	59.00-154.00 nmol/L	S/P	5.40-320.00 nmol/L	40 µL	15 min	CE
IF1067	fT3	Hyperthyroidism, hypothyroidism	3.10-6.80 pmol/L	S/P/WB	0.40-50.00 pmol/L	40 µL	15 min	CE
IF1068	fT4	Hyperthyroidism, hypothyroidism	12.00-22.00 pmol/L	S/P/WB	0.30-100.00 pmol/L	40 µL	15 min	CE

Cat. #	TEST ITEMS	DISEASES	CUT-OFF VALUE	SAMPLE TYPES	MEASURING RANGE	SAMPLE VOLUME	REACTION TIME	QUALIFICATION
Reproduction/Fertility								
IF1013	HCG+β	Fertility	5.1 mIU/mL	S/P	5.0-100000.0 mIU/mL	100 μL	10 min	NMPA CE
IF1055	LH	PCOS, infertility evaluation	Refer to User Manual	S/P	0.20-150.00 mIU/mL	100 μL	15 min	NMPA CE
IF1056	FSH	PCOS, infertility evaluation and pituitary disorders	Refer to User Manual	S/P	0.20-150.00 mIU/mL	100 μL	15 min	CE
IF1066	AMH	Fertility, PCOS, gonadal function, precocious/late puberty	Refer to User Manual	S/P	0.10-20.00 ng/mL	200 μL	15 min	CE
IF1048	PRL	Infertility, gonadal disorders	Refer to User Manual	S/P	0.50-200.00 ng/mL	100 μL	15 min	CE
IF1071	Prog	Infertility, evaluation of ovulation	Refer to User Manual	S/P	0.10-40.00 ng/mL	100 μL	15 min	CE
Tumor Markers								
IF1053	tPSA	Prostate cancer	4.00 ng/mL	S/P	0.50-100.00 ng/mL	100 μL	15 min	
IF1072	fPSA	Prostate cancer	1.00 ng/mL	S/P	0.05-30.00 ng/mL	100 μL	10 min	
IF1050	AFP	Liver cancer, cancer of ovaries or testicles, etc.	7.0 ng/mL	S/P	2.0-500.0 ng/mL	100 μL	15 min	CE
IF1051	CEA	Cancer marker: colon cancer etc.	4.7 ng/mL	S/P	2.0-500.0 ng/mL	100 μL	15 min	CE
Infectious Disease								
IF1057	Anti-HCV	Hepatitis C	1.00 S/CO	S/P	1.00-10.00 S/CO	100 μL	15 min	
IF1058	Anti-TP	Syphilis	1.00 S/CO	S/P	1.00-50.00 S/CO	100 μL	15 min	CE
IF1059	Anti-HIV	AIDS	1.00 S/CO	S/P	1.00-1000.00 S/CO	100 μL	15 min	
IF1064	HBsAg	Hepatitis B	1.00 IU/mL	S/P	1.00-100.00 IU/mL	100 μL	15 min	
IF1063	Anti-HBs	Hepatitis B	10.00 mIU/mL	S/P/WB	10.00-1000.00 mIU/mL	100 μL	15 min	
IF1084	2019-nCoV IgM/IgG	COVID-19	1.00 COI	S/P/WB		100 μL	10 min	CE
IF1091	SARS-CoV-2 Antigen	COVID-19	1.00 COI	Nasal swab/Saliva		100 μL	15 min	CE
IF1095	SARS-CoV-2 Neutralizing Antibody	COVID-19	Refer to User Manual	S/P/WB		100 μL	15 min	CE
IF1047	<i>H. pylori</i>	<i>H. pylori</i> infection	5.0 ng/mL	Stool	1.0-200.0 ng/mL	10-50 mg	10 min	CE
Others								
IF1077	Ferritin	Anemia/tumors	Male: 30.00-400.00 ng/mL Female: 13.00-150.00 ng/mL	S/P	0.50-1000.00 ng/mL	50 μL	15 min	CE
IF1069	Total IgE	Allergic disorders	Refer to User Manual	S/P/WB	1.00-2000.00 IU/mL	100 μL	15 min	CE

Coming Soon: FOB, ASO, RF, anti-CCP, Folate...



Address: 3/7, BIDD, Gorwa, Vadodara 390016 (INDIA)
Tel: +91-265-2281631
Email: mail@reckondiagnosics.com
Web:reckondiagnosics.com



Add: No.9 Bofu Road, Luhe District, Nanjing, 211505, China
Tel: +86-25-68568508/68568594
Fax: +86-25-68568500
E-mail: sales@getein.com.cn; overseas@getein.com.cn
Web: en.bio-gp.com.cn



FSC CE NMPA NGSP IFCC IVD

